

For Your Healthier Life

DOCTOR LIFE[®]

*Drainage, Circulation, Soft massage
for your beautiful life*




LX7



Doctor Life portable series The best choice for your happier and healthier life

All human bodies are managed by smooth circulation of blood and lymph.
Doctor Life systems provide dynamic drainage, circulation & soft pressure massage

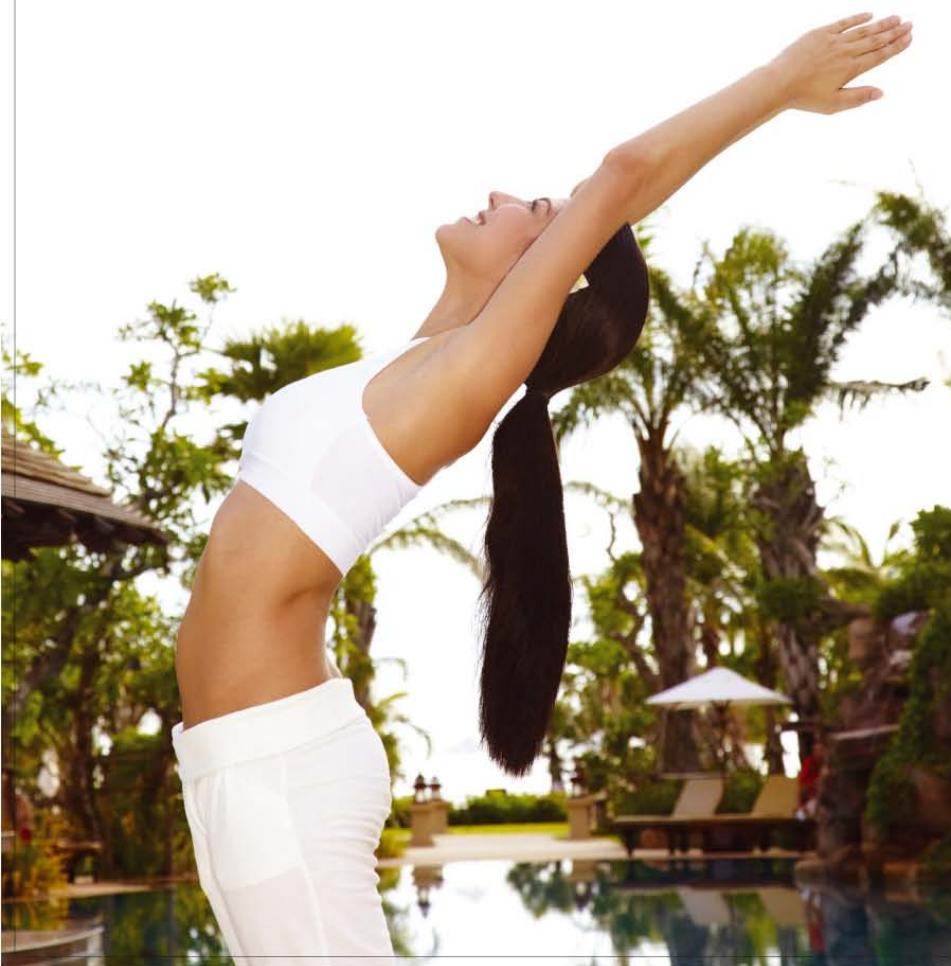

Luxury Portable, LX7 (4 chambers)

New concept, Digital Homecare System

LX7 (Automatic circulation system)

SYSTEM features

- Automatic operation button
- Touch key pad controllers
- Selection key from Massage mode and circulation mode
- Optional selection controller Foot/calf/knee/thigh
- Precious compression of a set pressure by a digital pressure sensor

Specifications

- Pressure range: 10 ~ 200mmHg
- Time range: 10, 20, 30 minutes
- 2-Compression mode: M(massage)mode, C(circulation) mode
- Optional care button: Foot/calf/knee/thigh
- Digital pressure sensor
- Machine size: 260 x 180 x 130mm
- Machine weight: 2kg



Products introductions


Total solution of air compression therapy solutions



LX7 (Luxury)

New automatic system

2 Compression modes

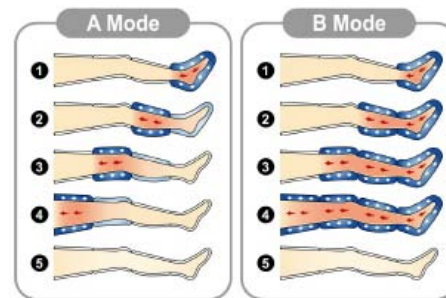

- Pressure: 20~250mmHg
- **Optional care** (foot, calf, knee, thigh)
- Automatic **optimum start button**
- **Soft touch key pad controllers**



Products introductions

Total solution of air compression therapy solutions



Products introductions


Total solution of air compression therapy solutions



Air Compression Therapy

Total solution of air compression therapy

- All human bodies are managed by smooth circulation of blood and lymph.
- Doctor Life system helps that various patients are recovered from their disease by promoting blood and lymph circulation and metabolism.
- Doctor Life system is consisted of air chambers and an air pump offering air compression massaging action to legs, arms or waist.

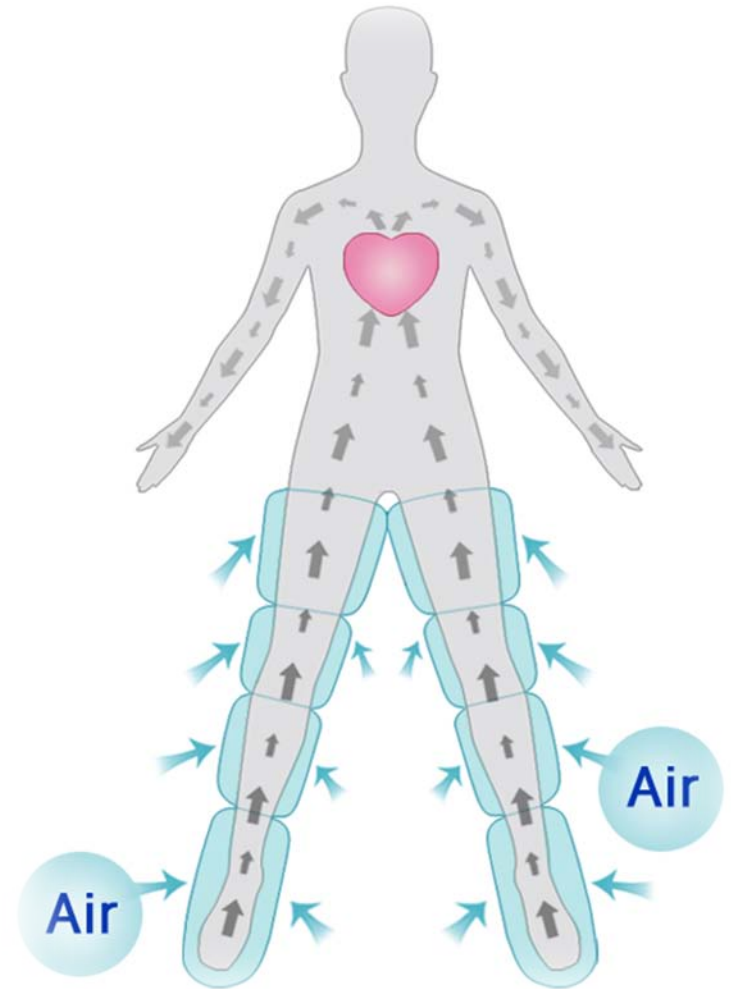

Air Compression Therapy

Total solution of air compression therapy

Medical Indications

- DVT (Deep Vein Thrombosis)
- Lymph edema
- Post-paralytic
- Limb convulsion
- Venosclerosis
- Diabetes
- Chronic venous insufficiency (CVI)
- Post-mastectomy
- Post-traumatic
- Spinal cord injury
- Varicose
- Sciatica
- Sprains
- Cellulite

Home care Indications

- Lymph edema
- Lymph
- Diseases by interruption in blood circulation
- Fat dissolution
- Pregnant woman's limb management
- Benumbed feet or hands
- Cold feet or hands
- Settlement of stress
- Exercise effects
- Old age, weakness and fatigue
- Insomnia
- For healthier body and elastic skin

Basic Functions for Home use

❖ Pump Function – (Deflation and Inflation Function)

Expanding of each capillary vessel by the pump function of the compression movement helps the delivery of clear blood including the increased oxygen over whole human body and the removal of the bad deposits (Cholesterol, etc.).

❖ Massage Effect

It massages fatigue and hard muscles, congestion parts or cellulite parts repeatedly with soft air pressure. It promotes the effects of exercise if it is used before and/or after exercise and it's like soft warming up.

❖ Stretching Effect

When cuffs pressurize legs, arms or waist, it makes the joints to be stretched naturally.

❖ Deep Breathing Effect (Abdominal Breathing)

It makes a user to breathe deeply unconsciously during using a device.

❖ Waist Cuff Effect

It is good for waist pain, intestine movement, dissolution of abdomen fat (cellulite) and pelvis contraction movement.

Home use Tips

- ❖ It is good to use Doctor Life with a patient's comfortable posture for the best effects.
- ❖ The pressure around 20~80mmHg for 15 minutes is proper, but the pressure intensity and time should be controlled by user's state and doctor's indications.
- ❖ We recommend all beginners to apply the device with the lowest pressure for 5 minutes for their safe use. User can increase the pressure intensity gradually, according to a patient's state and a doctor's advice.
- ❖ Use Doctor Life frequently for the best effects.
- ❖ We recommend to use only one arm cuff to prevent a heart attack and because the arm is near to the heart and a user can not use two arm cuffs at the same time. It's good to use two arm cuffs under doctor's indication and a protector to the near a user.
- ❖ Please breathe deeply and repeatedly by matching the rhythm of deflation and inflation of each chamber (Abdominal breathing) because the speed of Doctor Life is the similar with pulse speed. Please breathe out when air from each chamber comes out and breathe in when air comes in each chamber. It makes increase more the effects of Doctor Life.
- ❖ If Doctor Life is used before or after exercise or stretching, it helps the effect of it.

Sliming project for beautiful skin and body slim

1. Relaxation of body (Heat)

The relaxation of body is the warming up step to/before receives all therapies. To relax a body, thermo therapies like Sauna, Thermo therapy equipment use, Fomentation with hot pack, steam towel are used in this step. After a body have been warmed and relaxed sufficiently, please go to the next step, the circulation step.

2. Circulation of blood and lymph (Air compression)

After the body is sufficiently relaxed to start the body management, please apply this second step. The main purpose of second step is to help the blood and lymph of whole body to be circulated smoothly. Human can't get beautiful and healthy skin and body without active movement of whole body. Stagnant circulation and movement of body arises various disease and needless bad fat. So, this step applies/uses air compression like Suction therapy, Pneumatic compression therapy, etc. Specially, our Doctor Life is main item having superiority effects at this step.

“Doctor Life effects in this step”

✓ Cellulite (Orange haut) removal

Bad fat, cellulite arises usually at hip, thigh, abdomen, arm, etc around lymph grand having trouble lymph drainage. Doctor Life removes cellulite by circulating blood and draining lymph artificially with Doctor Life. Specially, as lymph vessel is so weak, the therapy of Lymph by soft air compression of Doctor Life is so effective. Now, Doctor Life is most suitable item to the circulation step.

Sliming project for beautiful skin and body slim

3. Bad fat dissolution

Doctor Life dissolves needless fat by increasing the temperature of body with regular circulation operation and massage of rigid muscle, etc.

4. Concentration management -

The step uses generally the therapies using Low frequency, Ultra sound, Supersonic wave, Needles

5. Maintenance

Exercise, Suitable dietary treatment, Doctor Life system, etc

Conclusion)

Esthetic management can't be completed, if only one step is handled carelessly or omitted.

Doctor Life can be used to promote the main purpose of each step but especially shows most prominent effects at the second step. At present, almost Korean esthetic shops use Doctor Life as main therapy equipment at the step of circulation and almost clients also are being satisfied with the therapy of Doctor Life.



Doctor Life is our representative brand of total solution of air applications by Daesung maref since 1986



We will do our best to be the No. 1 company of air compression systems in the world.

- Overseas sales manager, Helen Kim (ext.213)
- Eastern sales manager, Ryan Lee (ext.209)
- Western sales manager, Jay Kim (ext.235)

